

3M™ Reston™

Self-Adhering Foam Products



1

Self-adhering foam designed to distribute body weight to help prevent skin breakdown and pressure ulcers, and to prevent skin damage from splints, casts and prostheses. It can also be used as a padding for beds, chairs and operating room tables.

- Versatile padding adheres well to a variety of surfaces.
- Lightweight and conformable padding can be cut, layered and shaped, allowing it to be adapted for many applications.
- Protects against abrasions, shearing and pressure for increased skin protection and better patient comfort.
- Available in medium- and high-support pads or light support foam roll to meet your application needs.

3M™ Reston™

Self-Adhering Foam Products

Suggested Applications:

- Distributing body weight to help prevent skin breakdown and pressure ulcers.
- Preventing skin damage from splints, casts and prostheses.
- Use as a padding for devices like beds, chairs and operating room tables.

Contraindications:

This product is not to be used as a dressing in post-surgical treatment procedures such as liposuction.

Warnings:

Reston self-adhering foam products have not been evaluated for use as a compression dressing material in surgical procedures such as liposuction. The product should not be applied to the skin under tension. Such uses can result in damage to the skin such as blistering or tearing which may lead to hypopigmentation and the risk of serious infection. Reston self-adhering foam products should not be placed in direct contact with an open wound.

Ordering Information

3M™ Reston™ Self-Adhering Foam Pad

Cat. No.	Product Information	Size	Pads/Package	Packages/Case
	Medium Support:	7/16 in. thick		
1560M		7 7/8 in. x 11 3/4 in. 20 cm x 29.8 cm	10	5
	High Support:	15/16 in. thick		
1561H		7 7/8 in. x 11 3/4 in. 20 cm x 29.8 cm	5	5

3M™ Reston™ Self-Adhering Foam Roll

Cat. No.	Product Information	Size	Rolls/Package	Packages/Case
	Light Support:	3/16 in. thick		
1563L		4 in. x 196 in. 10 cm x 5 m	1	5

Contact your 3M Health Care Sales Representative, or call the 3M Health Care Customer Helpline at **1-800-228-3957**. These products can be ordered from your local distributor. Outside the United States, contact the local 3M subsidiary.



Health Care

3M Center, Building 275-4E-01
St. Paul, MN 55144-1000
USA
1-800-228-3957
healthcare@3M.com
www.3M.com/healthcare

3M Canada

Post Office box 5757
London, Ontario N6A 4T1
Canada
1-800-563-2921



Recycled Paper
40% pre-consumer
10% post-consumer

Printed in U.S.A.
© 3M 2001, 2002 70-2008-0838-7 (102.2) DPI